



Titrating the EMA appliance

The EMA oral appliance from Myerson increases airway space by advancing the mandible using interchangeable elastic straps. EMA elastic straps come in 9 different lengths (see figure A), and 4 different strengths (see figure B). The shorter the ema elastic strap, the farther the mandible is advanced (see figure C). 36 different straps allow for the most effective repositioning with maximum comfort.

Figure A 9 different lengths allow for 1 mm advancement of the mandible


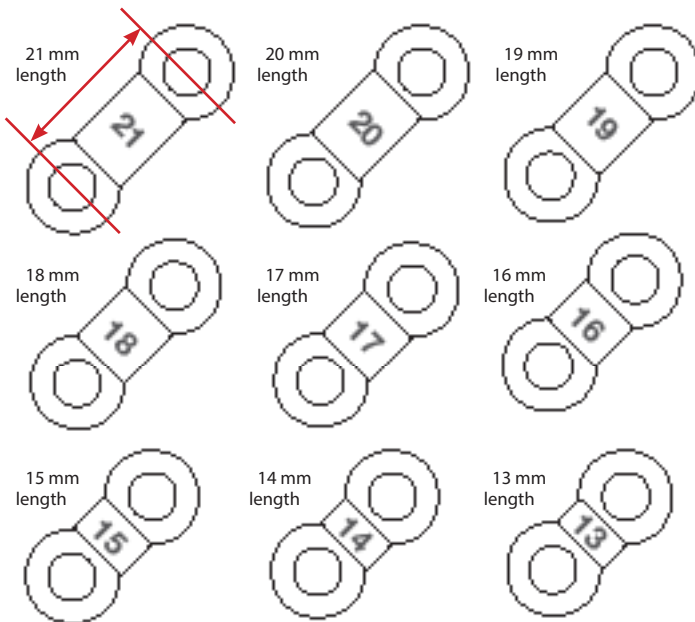


Figure B Each EMA elastic strap comes in 4 different strengths, indicated by color.

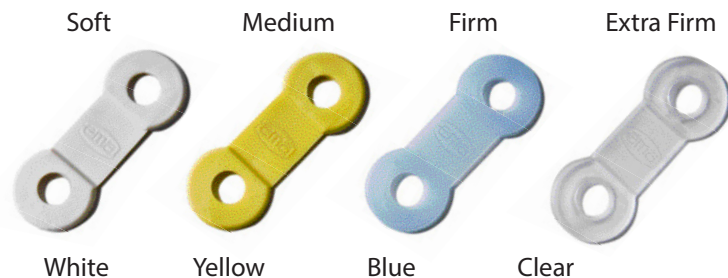


EMA elastic straps are 100% latex free.

The shorter the EMA elastic strap, the farther the mandible is advanced.

Figure C

EMA buttons are 27 mm apart*

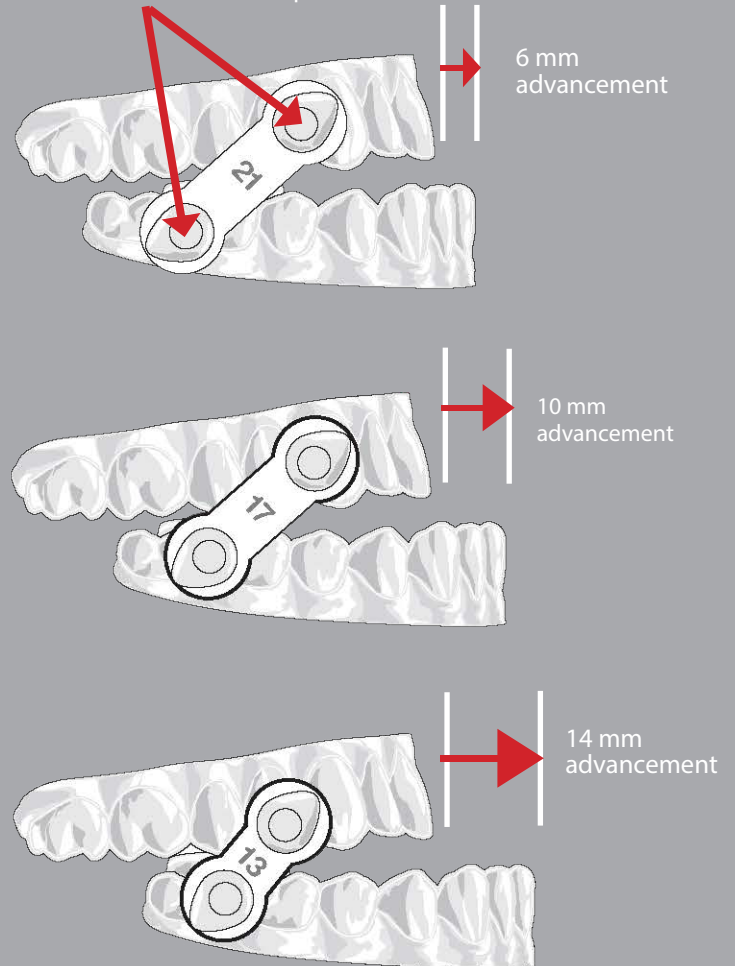

EMA* STRAP LENGTH	AVERAGE AMOUNT OF ADVANCEMENT
21 mm	6 mm
20 mm	7 mm
19 mm	8 mm
18 mm	9 mm
17 mm	10 mm
16 mm	11 mm
15 mm	12 mm
14 mm	13 mm
13 mm	14 mm